

PREMIUM ALGINATE SYSTEM

CAVEX GREENCLEAN

-

The smart solution for removing alginate
and gypsum residue.



YOUR
IMPRESSION
IS OUR
SPECIALTY

CAVEX

PREMIUM ALGINATE SYSTEM

CAVEX GREENCLEAN

Who would be better to produce an alginate dissolver than the makers of "The Best Alginates in the World"? Cavex GreenClean proves this. Thanks to the special formula, impression trays, spatulas and instruments can be cleaned quickly and easily.

The secret of Cavex GreenClean

Removing alginate or gypsum from impression trays and instruments can be a tedious task. Scratching or sanding is not only time-consuming, but can also cause permanent damage to impression trays and instruments. With Cavex GreenClean, this is a thing of the past. The formula has been specifically developed to break down the organic matrix of the alginate or gypsum which completely dissolves any residue.

Application

Add two scoops of Cavex GreenClean to half a liter of lukewarm water. Remove any loose residue and immerse the impression trays and instruments for 15-30 minutes. Just rinse with water and the impression trays and instruments are clean and ready for use again. With warm water or an ultrasonic bath, the process is even faster.

Features

- **Fast, effective and easy to use**
- **Refreshing citrus smell**
- **Biodegradable**
- **Skin friendly**
- **pH neutral**
- **Economical, 1 kg cleans 500+ trays**

More than 500 instruments can be cleaned with one jar of Cavex GreenClean. In addition to effectiveness and convenience, this product is therefore also very economical to use.

pH neutral and environmentally friendly

Cavex GreenClean only reacts with the alginate or plaster residues, and not with the trays and instruments themselves. These remain intact and unaffected, just like the skin. The solution is pH neutral and 100% biodegradable. Therefore, it can simply be disposed of in the sink after use. Easy and responsible, that's what Cavex GreenClean is all about.



YOUR IMPRESSION IS OUR SPECIALTY

Fustweg 5 | 2031 CJ Haarlem | The Netherlands | cavex.nl

CAVEX

ZYH001E.03