Tedequim

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FAgamin[®]

PÁGINA..... 2; 3

INSTRUCTIONS FOR USE

FAgamin[®]

PAGE..... 4; 5

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FAgamin

Silver Diamine fluoride 38%

Cariostatic, bactericidal, remineralizing and dental desensitizing.

FAgamin® contains a 38% solution of silver diamine fluoride ($\text{Ag}(\text{NH}_3)_2\text{F}$). Its action on teeth is based on a chemical reaction with hydroxyapatite, forming insoluble deposits on enamel and dentin. Thus, silver diamine fluoride acts by occluding dentinal tubules with deposits containing silver, calcium, phosphate, and fluorides, reducing tubular diameter and blocking stimulus transmission, thereby reducing dental sensitivity.

The deposits also act on the tooth surface, reinforcing and protecting it, which explains FAgamin®'s remineralizing effect.

Silver ions (Ag^+) inhibit bacterial growth and possess enzymatic action. Ammonia stabilizes silver ions by forming a complex that keeps them in aqueous solution, reaching a basic pH. Fluoride ions decrease hydroxyapatite solubility and inhibit further caries development.

Indications of use:

FAgamin® has a wide range of clinical applications due to its chemical composition, including enamel remineralization in older teeth, remineralization of dentin adjacent to caries, caries detection, inhibition of incipient or established caries, and desensitization of dental hypersensitivity. Additionally, it exhibits bactericidal and antiseptic properties. FAgamin® provides a quick and simple topical application method for treating cavities in primary teeth, without anesthesia and painlessly.

Applications:

It is applied in temporary teeth, in incipient or rampant caries, in pits and fissures (after widening if deemed necessary by the dentist), in cavity preparations for prosthesis, crowns, dies, fixed bridges. It can be applied in root canals diluting it (1:10 with distilled water).

Recommended for patients with severe cognitive or physical disabilities and dental phobias.

Contraindications:

Do not use in patients with:

- 1) Ulcerative gingivitis,
- 2) Known sensitivity to any components of this product.
- 3) Patients who have undergone complete mouth gingivectomies.

Instruction for use

1. Avoid contact of FAgamin® with the mucous membrane; protect with cotton rolls or rubber dam.
2. Isolate gingival tissue with Gingival and Labial Protector TEDEQUIM®, vaseline or cocoa butter.

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3. Wash and dry the tooth before starting treatment.
4. Place 2 or 3 drops of FAgamin® in a disposable container or silicone Dappen dish. (Do not use glass containers).
5. Apply FAgamin® using a swab or brush soaked in 2 or 3 drops of the product. Apply to the treatment area of the tooth, leave for a few minutes, rubbing the applied area.
6. Finally, remove excess with a swab. Do not rinse.

For incipient caries, apply FAgamin® only once; for other cases, multiple applications may be necessary, spaced one week apart.

Limit desensitization applications to no more than 5 sites per patient.

Warnings:

- 1) The product should be used for topical and local application only.
- 2) Do not swallow.
- 3) keep out of reach of children.
- 4) Protect the patient's eyes and clothing.
- 5) In case of eye exposure, rinse open eyes with running water and seek medical attention immediately.
- 6) In case of skin contact, wash immediately with plenty of water.
- 7) Avoid contact with gums. When contact occurs accidentally or when the patient manifests pain, , rinse mouth immediately with saline or 10 V hydrogen peroxide.

Precautions:

- FAgamin® generally does not stain enamel but may darken soft dentin or composite restoration margins. Staining can be reduced depending on cavity depth and location by using white fissure sealant, glass ionomer, or opaque resin.
- Caution during clinical handling to prevent product contact with the gingival margin. Improper isolation may cause discomfort or slight irritation.
- Do not apply when there are known deep seepages.
- Inform patients that air-drying and product application may cause transient discomfort in hypersensitive areas.

In the case of deep cavities, since filtrations to the pulp may occur, consider avoiding use or diluting the dose with distilled water (1:10).

Handling care:

- Immediately seal container after use.
- Wear gloves.
- Store in a light-protected place.
- If skin is stained, cleanse with household bleach using cotton or 10 V hydrogen peroxide.
- In case of spillage on any surface, use sodium hypochlorite immediately.

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- Do not change bottle original.

Adverse reaction:

Transient gingival irritation may rarely occur.

Composition:

38% W/V Silver Diamine Fluoride Solution, $\text{Ag}(\text{NH}_3)_2\text{F}$.

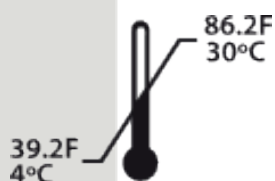
Presentation:

5 ml black plastic dropper bottle.

Storage:

Keep at temperatures between 4° and 30° C, well closed and protected from light.

Do not freeze and keep away from heat sources.

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